



ELITE-SPK

**TRANSOSSEOUS
I M P L A N T
F O R T H E
A R T H R O S C O P I C
R E P A I R O F
R O T A T O R
C U F F
L E S I O N S**



INNOVATIVE

Oper platform.

VERSATILE

For arthroscopic and mini-open use.

SAFE


Prevents the bone cutting phenomenon.

The implant Elite-SPK allows the repair of rotator cuff tears with transosseous technique, in total surgical freedom, as it is possible to choose between arthroscopic or mini-open approach, both applicable without incurring the phenomenon of “bone cutting”.

A single Elite-SPK implant creates a contact surface between the tendon and the bone that provides biomechanical effectiveness comparable to the gold standard techniques:

- The peculiar shape of Elite-SPK and its features make it a suture platform that is perfect for the transosseous approach. Allows the application on very fragile bone tissue and eliminates the phenomenon of migration and pull out.
- It also allows the wide decortication of the footprint, while still providing maximum seal bone.
- The special geometry of Elite-SPK is designed to use up to 3 sutures on the same device, which is useful in order to equally distribute the forces on the more worn tendons.

SOME SUTURES CONFIGURATIONS WITH ELITE-SPK

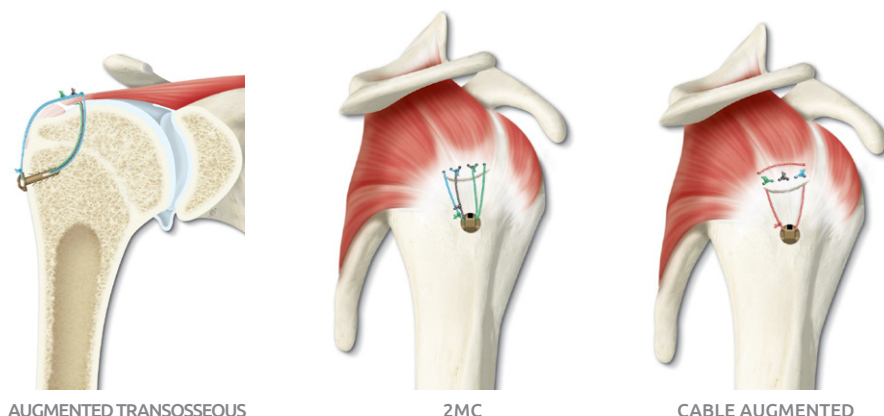


SPEED OF INPLANT AND DEDICATED INSTRUMENT


PERFORM THE TRANSOSSEOUS TUNNEL WITH THE DEDICATED INSTRUMENT..



LOAD THE SUTURES ON THE ELITE-SPK AND PLACE THE DEVICE ON THE DEDICATED INSERTER.

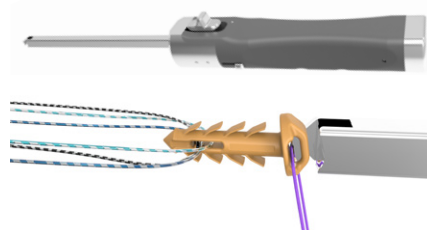

PUSH ELITE-SPK INTO THE TRANSOSSEOUS HOLE AND RELEASE IT FROM THE INSERTER, CHOOSE THE MOST APPROPRIATE SUTURE CONFIGURATION AND REALIZE THE REPAIR.

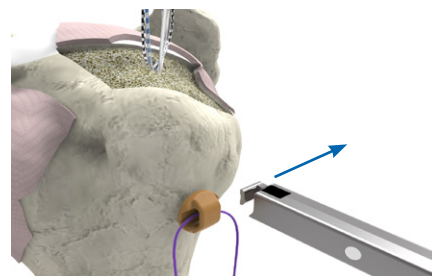

| CODE | CND | RDM | DESCRIPTION | Q.TY | PACKAGING |
|----------------------------|-----------|---------|--|------|------------------------------|
| C000508 | P09120102 | 1250679 | Elite-SPK | 1 | Sterile γ Rays |
| A000115 | / | 102103 | Elite-SPK KIT: 1 x Elite-SPK (cod. C000508) 3 x Sutures HS Fiber USP 2-Metric 5* No needed (cod. HSe-2G/2C/2K-NN) | 1 | Sterile γ Rays Sterile EO |
| P005_AS034_13 | K0399 | 1220833 | Inserter Elite-SPK | 1 | Non sterile |
| CLASS OF RISK OF ELITE-SPK | | IIB | | | |
| MATERIAL OF ELITE-SPK | | Peek | | | |

- See product’s data sheet for further information.



NCS Lab Srl.
Via Pola Esterna 4/12 - 41012 Carpi (MO) Italy
pIVA 025500410202 | Tel. +39 059 669813 | biomed@ncs-lab.com